

HYCOAT[®]

Topical Wound Management Solution

Hyaluronate sodium sterile solution

Sodium hyaluronate/hyaluronic acid (HA) plays an important role in the wound healing process. Sodium hyaluronate acts as a lubricant on the skin by restoring moisture and aiding in prevention of friction or abrasion. It is a polysaccharide that is distributed widely in the extracellular matrix of connective tissue. HYCOAT[®] helps manage surgically induced and chronic wounds by providing a moisture rich environment and acts as a tissue coating. HYCOAT[®] protects the wound bed by the formation of an occlusive gelatinous barrier, reducing the chance for infection to occur. HYCOAT[®] helps to reduce pain at the wound site by occluding nerve endings that signal pain, conforms to any wound site and is easy to handle and deliver.

Description:

HYCOAT[®] is available in a 6 mL solution containing 30 mg of sodium hyaluronate, a 10 mL solution containing 50 mg, and a 2mL solution containing 20mg of sodium hyaluronate having a pH between 6.5-7.25 and is preservative free. The average relative molecular weight of HYCOAT[®] exceeds 1.0 million Daltons. HYCOAT[®] is a highly consistent product.

Indications:

HYCOAT[®] is indicated for use in the topical management of chronic and acute wounds including:

- ❖ Pressure ulcers (Stages I-IV)
- ❖ Acute wounds (such as cuts, abrasions, and lacerations)
- ❖ First and Second Degree Burns
- ❖ Surgical wounds (including donor sites)
- ❖ Venous stasis ulcers
- ❖ Autograft procedures

Veterinary device is for dogs, cats and horses (not for food animal use).

Precautions and Contraindications:

Discard unused portion. HYCOAT[®] is to be dispensed by prescription only.

Federal law restricts this device to sale by or on the order of a licensed veterinarian (in animal marketplace).

Product Administration:

Using HYCOAT[®]:

1. Cleanse wound/surgical site.
2. Withdraw the HA solution.
3. Dispense topically onto the wound site.
4. Cover wound area with a non-adherent dressing.
5. Change dressing as needed, repeating application of HYCOAT[®].

*Wounds that are contaminated and/or infected should be debrided prior to using the HYCOAT[®] product.

Storage:

- ❖ Store at temperatures between (15° – 30° C); Protect from freezing; Sterile when unopened

The Hymed Group Corporation, 1890 Bucknell Drive, Bethlehem, PA 18015

The information given herein is believed to be reliable. However, no guarantee is made or liability assumed. The Hymed Group Corporation does not know all of the uses to which its products may be utilized or the condition of use. Therefore, The Hymed Group Corporation makes no claims or warranty concerning the fitness or suitability of the product for a particular use or purpose other than those outlined.

MADE in USA

7/12