

**Instructions for Use**  
(Please read carefully before using)

**PATENTED**

## **hyCURE® ORAL GEL**

**Medical Hydrolysate of Type I Collagen**  
***Advanced Wound Care***

hyCURE® ORAL GEL is a natural gel. hyCURE® ORAL GEL interacts with the wound site maintaining a moist environment that encourages wound healing. A revolutionary breakthrough in advanced woundcare, hyCURE® ORAL GEL is the *only product of its kind*. It contains only a natural medical hydrolysate, Type I collagen as its active ingredient. No other preservatives or potentially harmful ingredients are present.

### **Indications:**

Denture Sores , Oral Ulcers (Non-infected and Viral), Periodontal Surgical Wounds, Extraction Sites, Suture Sites, Traumatic Wounds, Burns, and Surgical Wounds.

### **Characteristics:**

- ❖ Protects the wound bed and newly formed granulation tissue by the formation of an occlusive gelatinous barrier
- ❖ Reduces pain
- ❖ Conforms to any wound site
- ❖ Biocompatible , biodegradable all natural product
- ❖ Safe to use, non-toxic
- ❖ Soothes and deodorizes
- ❖ Promotes natural autolysis by rehydrating and softening necrotic tissue and eschar, thereby encouraging autolytic debridement
- ❖ Easy to handle and deliver
- ❖ Absorbs wound exudate

### **Precautions and Contraindications:**

- ❖ No known side effects
- ❖ Non-toxic
- ❖ No contraindications
- ❖ If condition worsens or does not improve within 10-14 days, consult a physician.

Keep out of reach of children

### **Storage:**

- ❖ Store at controlled room temperature
- ❖ Protect from freezing
- ❖ Extreme temperatures may affect viscosity, but will not affect performance of product

### **Product Administration:**

1. Cleanse the wound site using normal saline.
2. Apply a thin layer of hyCURE® ORAL GEL directly into the wound and onto the surrounding area.
3. Allow to air dry then cover wound with a non-adherent dressing. Gauze is recommended as the secondary dressing if removing debris from wound site. Superficial wounds may not require a secondary dressing due to the film that hyCURE® ORAL GEL forms on the wound site after drying.
4. Remove dressing as needed (if secondary dressing sticks to wound, soak in warm water to ease removal) and repeat procedure.

### **How Supplied:**

hyCURE® ORAL GEL is available in ¼ oz (6.0g) and 1 oz (28g) tubes.

**Hymed®, 1890 Bucknell Drive, Bethlehem, PA 18015**  
**PHONE: (610) 865-9876 FAX: (610) 691-5930**

The information given herein is believed to be reliable. However, no guarantee is made or liability assumed. Applied Nutritionals does not know all of the uses to which its products may be utilized or the condition of use. Therefore, Applied Nutritionals makes no claims or warranty concerning the fitness or suitability of the product for a particular use or purpose other than those outlined.

02/05