



Description

Chondroprotec is a sterile 10 mL solution containing 1000 mg of Polysulfated Glycosaminoglycan (100 mg/mL). For use on dogs, cats and horses.

Indications

- Partial and full thickness wounds
- Secreting and bleeding dermal lesions
- Trauma injuries
- 1st and 2nd degree burns
- Pressure ulcers (Stages I-IV)
- Surgical incisions
- Donor sites
- Venous stasis ulcers

Features

- 1000 mg of PSGAG in a 10 mL vial (100 mg/mL)
- Derived from bovine trachea
- Preservative free
- Patented technology

Directions for Use

1. Irrigate wound with normal saline solution.
2. Either pour contents of vial or dispense from syringe to withdraw solution and cover wound site.
3. Cover with a non-adherent dressing such as PUF (polyurethane film) or Telfa[®]. *Note: Subsequent applications of Chondroprotec can be injected through the dressing as to not disturb the wound bed.*
4. Reapply Chondroprotec daily and redress wound site every 5-7 days or as needed.

Caution

Not for food animal use.

No known side effects, not intended for use as a long term or permanent dressing.

Federal law restricts this device to sale by or on the order of a licensed veterinarian.

Storage

Store at controlled room temperature (15-30°C or 59-86°F).

Protect from freezing.

Sterile when unopened.

Discard unused portion.

Presentation

#08820 1000 mg/10 mL vial
3 vials/box

R_x

